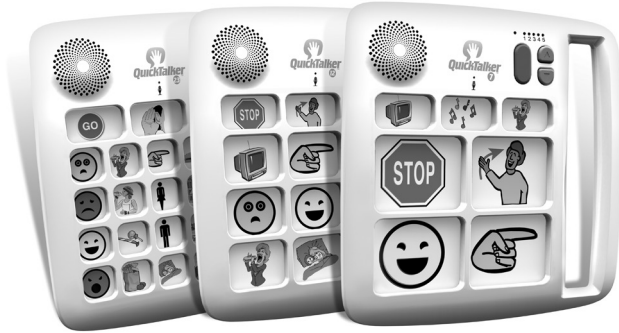
## Quickstart Guide

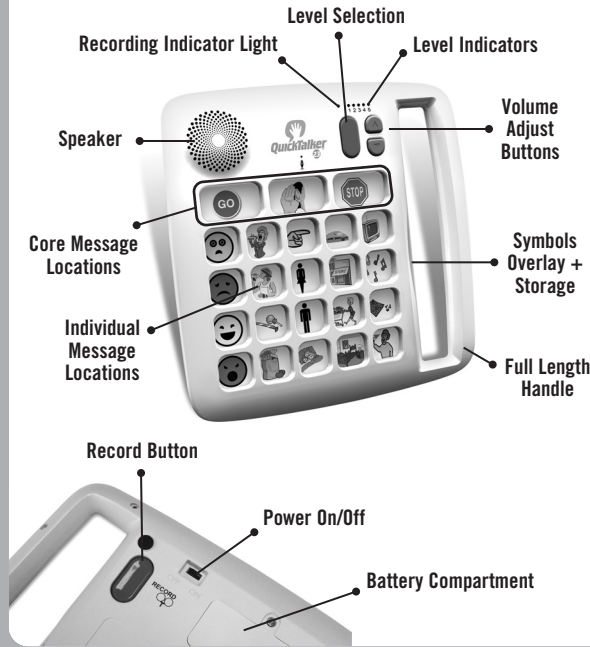
QuickTalker™ 7  
QuickTalker™ 12  
QuickTalker™ 23

AbleNet®

www.ablenetinc.com


## Key Features

## Step 1 - Powering QuickTalker™

### Powering QuickTalker™

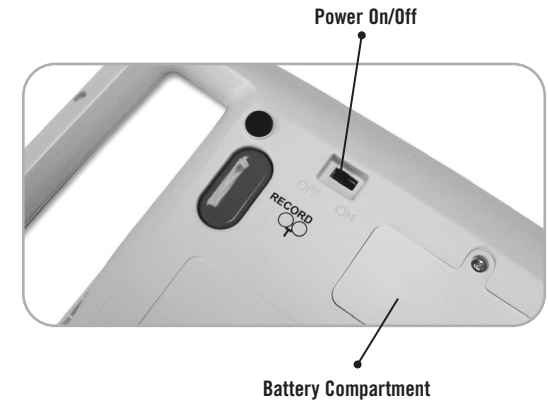
Turn the device to the back side. Insert 4 AA batteries into the device where shown. Use the On/Off switch to power the device.

## Step 2 - Select Level

### Select Level

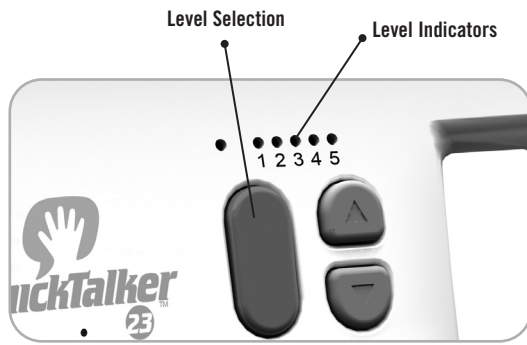
Select LEVEL (1-5) by pressing the red button on the front of the device.

## Step 3 - Recording Messages

### a) Entering Record Mode

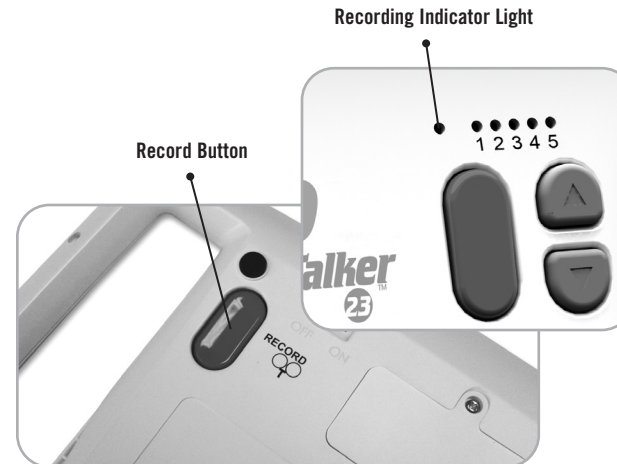Press and hold the RECORD button on the back of device until the red light is on, then release.

### b) Recording Messages

Press and hold the desired message location, begin speaking after the beep, then release. Repeat for other locations.

**Tip!** Speak 6-8" from the microphone for loud, clear sound.

### c) Exiting Record Mode

Press and release the RECORD button. Lights turn off. Note: The device will automatically exit Record Mode after 20 seconds of no activity.

## Step 4 - Playing a Message

### Playing a Message

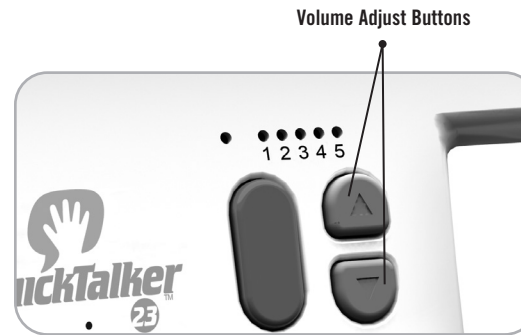
Press the desired location to play a message.

**Note:** Pressing another location will stop the current message.

### Adjusting Volume

Press the gray arrow buttons to adjust the volume.

### Message and light function

When a location is pressed the record light will flash yellow if the location is blank; red if recorded.

## Erasing Messages

**Single Location** - Enter Record Mode, then quickly press and release the location.

**Level** - Press and hold RECORD button for 10 seconds and all messages on current level will be cleared. Core messages will remain.

**Master Clear** - Press and hold RECORD button for 30 seconds and ALL QuickTalker messages will be cleared.

Record Button

## Advanced Features

### Core messages

The three core messages will be the same for all levels. The last recorded messages will be played.

### Locking the Controls (RECORD & LEVEL)

Press and hold the red RECORD and LEVEL buttons simultaneously for 5 seconds. All lights will flash twice. Repeat to unlock controls.

Record Button

Level Indicators

## Warranty and Care

### Cleaning and Care

To clean wipe with a slightly damp cloth.

### Warranty

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text is available at [www.ablenetinc.com](http://www.ablenetinc.com).

### More Information

You can find warranty, guarantee and return/repair information online.

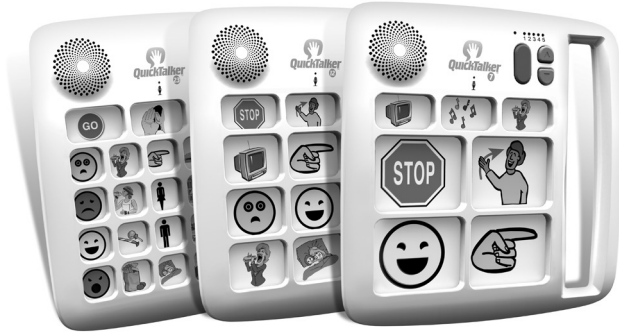
For more information on AbleNet products and services, or for contact:

### AbleNet, Inc.

Mpls./Saint Paul, MN 55113  
800-322-0956  
[www.ablenetinc.com](http://www.ablenetinc.com)

## Guide de mise en route rapide

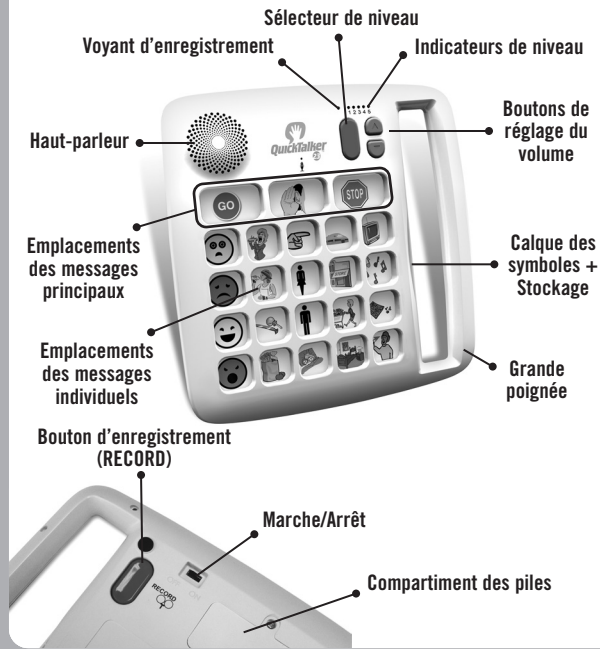
QuickTalker™ 7  
QuickTalker™ 12  
QuickTalker™ 23

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